

Accidental Deployment/Off-Loading

2


Issue: Anchor becomes detached from Pusher

Cause:

- Pressure on the white strike-cap, or
- Surgeon withdraws the Delivery Tool to remove more bone (to achieve a better angle or position).



Prevention:

- Emphasize 'in with the blue, out with the white' to the surgeon
- Ensure that the scrub-tech knows to only hold the Delivery Tool by the blue handle.

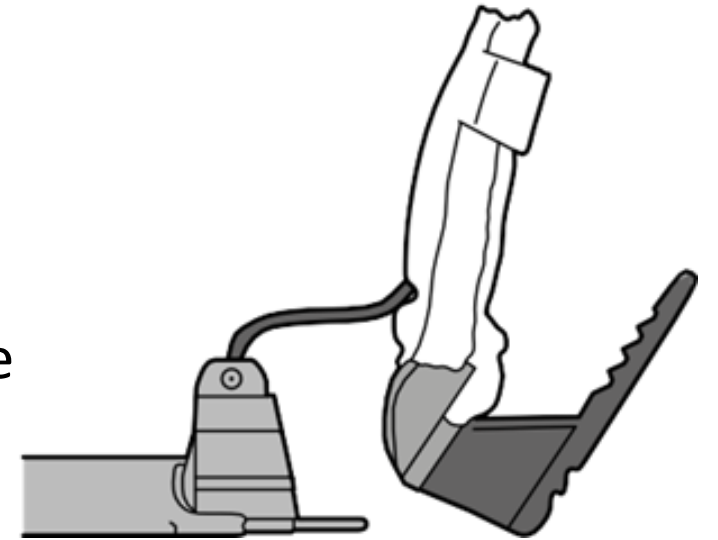

Do not touch the strike-cap


- Only hold the Delivery Tool by the **blue handle** – do not touch the white Strike Cap

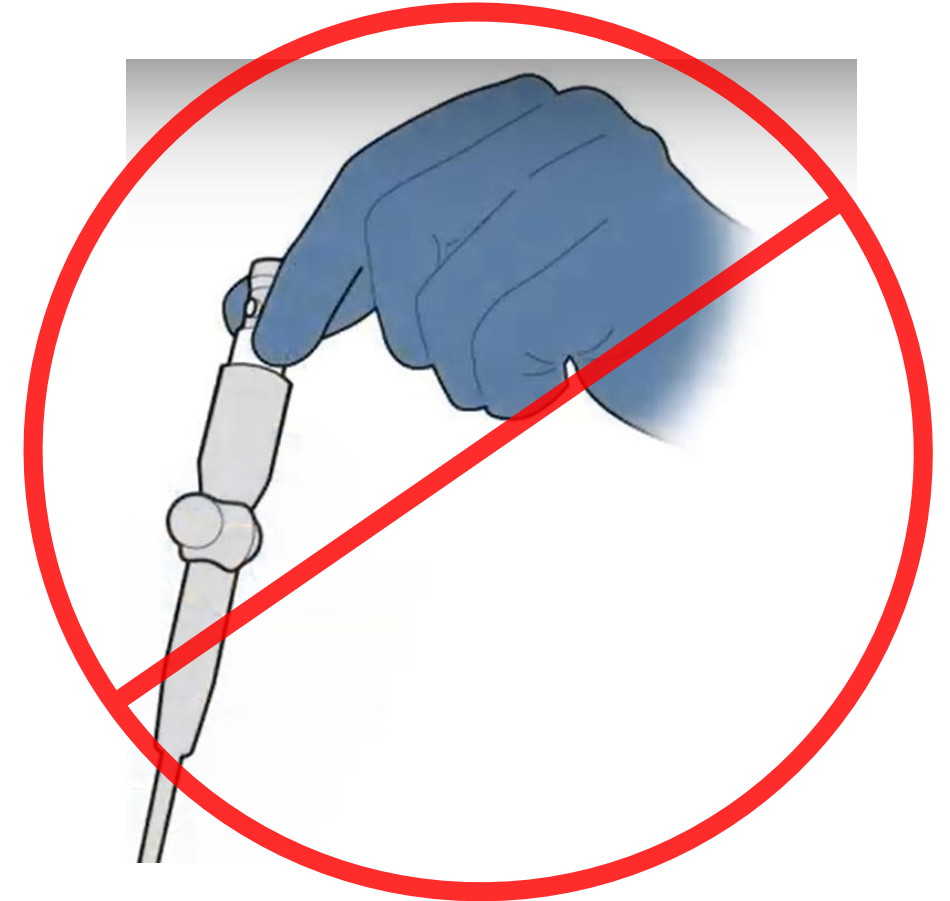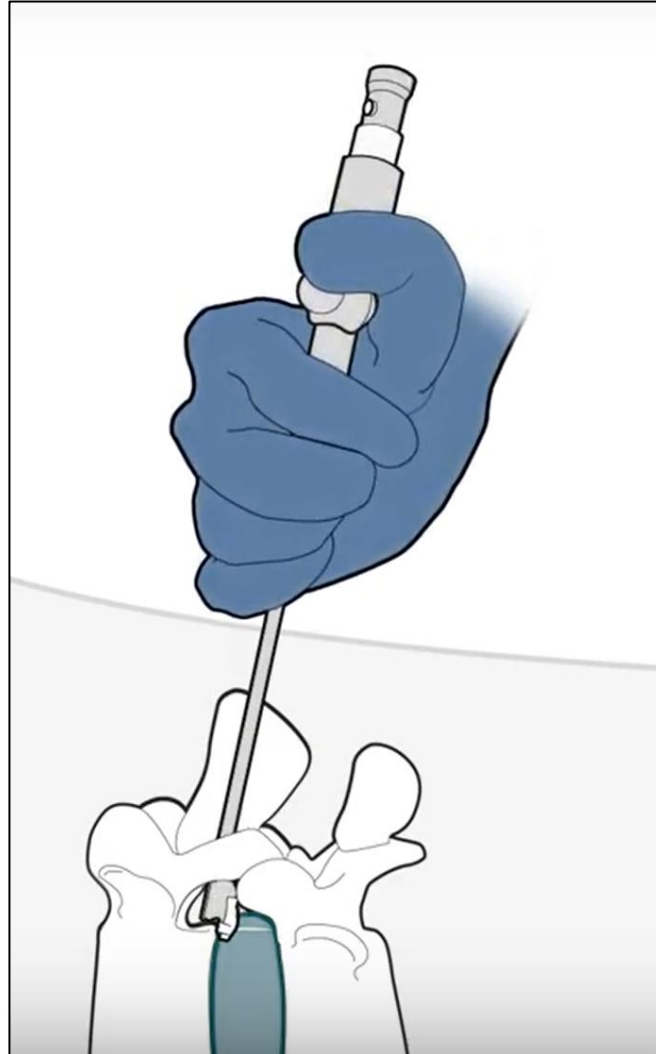

In with the blue, out with the white

- Only hold the Delivery Tool by the **blue handle** – do not touch the white Strike Cap
- The only time you should touch the Strike Cap is **if you have to remove the Delivery Tool to take out more bone...** then pull it out by the white (but don't touch the blue)

**"In with the blue,
out with the white"**

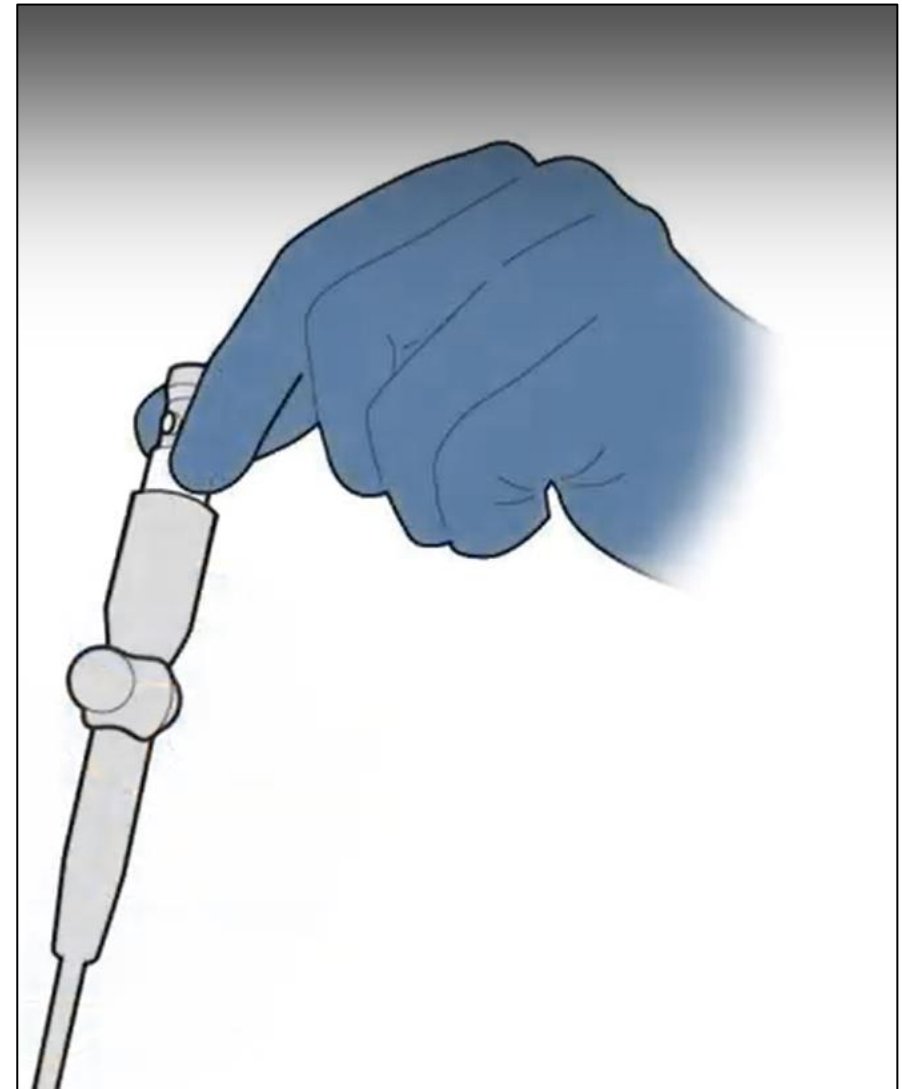

Accidental Deployment/Off-Loading

Solution:

- Always carry a loaded delivery tool into surgery to demonstrate reloading
- Explain how to reload the implant, and/or show this video, and see if they can reload the device
- If they can't reload, open a new implant
- Comp the wasted one or charge for it if you feel it was avoidable

